



Clearly different.
Clearly better.



Clearview EASY HCG

A one-step test for the rapid, accurate and
highly sensitive detection of hCG in urine

Early detection of pregnancy reduces the risks to mothers and their babies by:

- Altering behaviour, such as reducing alcohol or stopping smoking
- Increasing the intake of folic acid for a healthier pregnancy
- Avoiding foods, infections or treatments that could cause miscarriages or birth defects

Features

- Results in 3 minutes
- 4 sampling methods
- Sensitive to 25mIU/ml
- Proven accuracy of >99%
- Inbuilt procedural control
- Just one-step
- Sealed unit after use
- Room temperature storage
- Long shelf life

Benefits


- Rapid
- Flexible
- Ability to detect pregnancy on day of missed period
- Reliable
- Confirmation that the test has been performed correctly
- Easy to use, minimal training required
- Hygienic
- Ready to use, leaves valuable refrigerator space free
- Small volume users need not worry that product will expire before used

Clearview EASY HCG

HCG detection made simple


I Use one of the following 4 sampling procedures.

A Dip

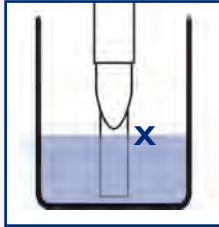


- Dip absorbent sampler into sample to point X for 15 seconds.
- Place cap over absorbent sampler, without inverting device.
- Read result after 3 minutes.

B Dip and leave

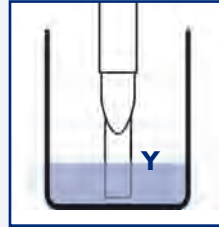

- Dip absorbent sampler into sample until it is half immersed (Y).
- Leave for 3 to 10 minutes.
- Place cap over absorbent sample without inverting device.
- Read result immediately.

C Pipette

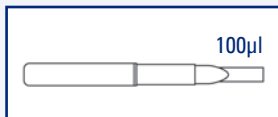

- Place on a clean, dry, flat surface with absorbent side of sampler facing upwards.
- Pipette 100µl of sample onto absorbent sampler.
- Place cap over absorbent sampler, without inverting device.
- Read result after 3 minutes.

D Urine Stream




- Absorbent sampler should be held in urine stream for 3 to 7 seconds **only**.
- Place cap over absorbent sampler, without inverting device.
- Read result after 3 minutes.

2 A blue Control Line must appear within 3 minutes otherwise the test must be repeated. Any result that appears after 10 minutes should be ignored.

Control Line

Result Line

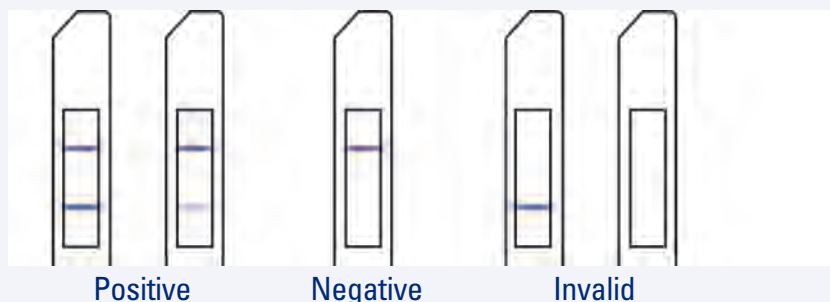

Positive

Negative

Invalid

See package insert for complete instructions.



**Inverness Medical has
changed it's name to Alere**

0161 483 5884
ukcustomer@alere.com
www.alere.co.uk

Product Information

Cat No: 502575
(English, Spanish,
Portuguese, Greek)

Cat No: 502922
(French, Italian,
German, Dutch)

Cat No: 504460
(Norwegian, Swedish,
Finnish, Danish)

Kit Size 20 Tests

Shelf Life 36 Months

Storage 2 - 30°C

Sample Type Urine

Time to Result 3 minutes

4 Sample Procedures Dip
Dip and leave
Pipette
Urine stream

Sensitivity **25mIU/ml**

Specificity **100%**

Accuracy **>99%**


Clearview

Clearly different.
Clearly better.

www.clearview.com