



# SureStep™

## One-Step hCG Pregnancy Test

Be confident... Be definite... Be sure



### SureStep™ hCG

is a simple test for the detection of hCG in urine as an aid to early confirmation of pregnancy




SureStep™ hCG provides a clear positive or negative result in just 3 minutes

### Features and benefits

- Rapid
- Sensitive to 25mIU/ml
- Just one step
- Proven accuracy of >99%
- Inbuilt procedural control
- Room temperature storage
- Long shelf life
- Results in 3 minutes
- Detects pregnancy on day of missed period
- Minimal training required
- Reliable
- Confirmation of correctly performed test
- Ready to use. No need to refrigerate
- Helpful for small-volume users

[www.suresteptest.com](http://www.suresteptest.com)

 inverness medical uk




# SureStep™

## One-Step hCG Pregnancy Test

### Test Procedure

- 1 Using the transfer pipette, apply 3 full drops of urine to the specimen well.
- 2 The test must be read 3 minutes after applying the sample. A pink/red line must appear in the control line region (C) to show that the test has worked correctly. Any result that appears after 10 minutes should be ignored.

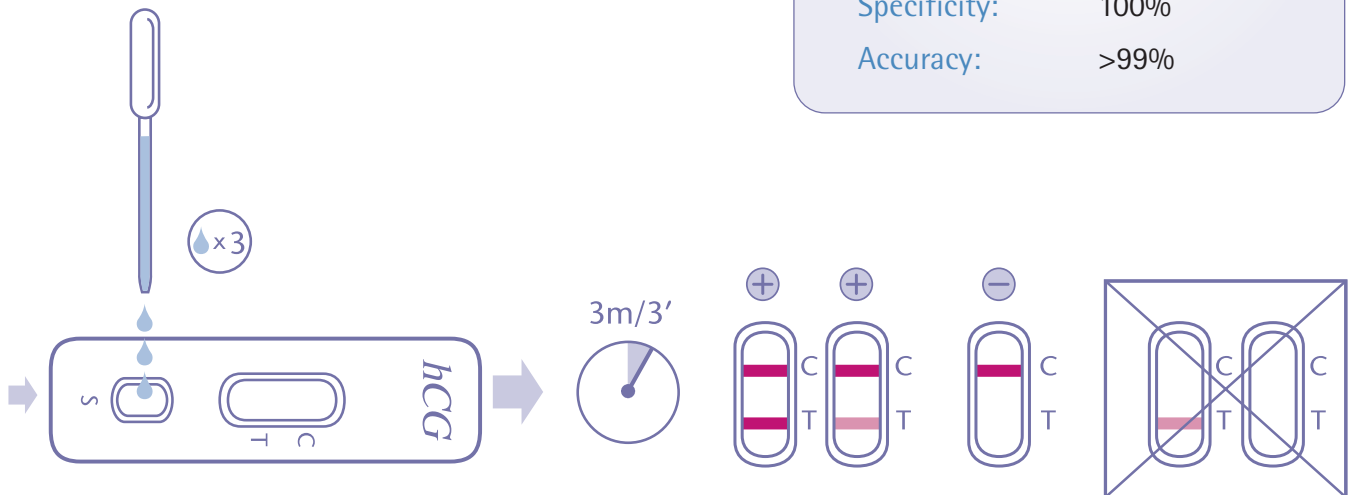

### Product Information

Cassette cat no.	FHC-102
Kit size:	40
Shelf life:	24 months
Storage:	2 - 30°C
Sample type:	Urine
Time to result:	3 minutes
Sensitivity:	25 mIU/ml
Specificity:	100%
Accuracy:	>99%

Distributed by



**Inverness Medical has changed its name to Alere**  
 0161 483 5884  
 ukcustomer@alere.com  
 www.alere.co.uk

### Further Products

- SureStrip hCG FHC-101
- SureStep LH FLH-102
- SureStep Drug Screen Test (DST)

[enquiry@surestep.com](mailto:enquiry@surestep.com)

Unipath Limited, Bedford MK44 3UP, United Kingdom  
 Tel: +44 (0)1234 835000 Fax: +44 (0)1234 835009

