



## One-Step hCG Pregnancy Test

Be confident... Be definite... Be sure



is a simple test for the detection of hCG in urine as an aid to early confirmation of pregnancy



SureStrip™ hCG provides a clear positive or negative result in just 3 minutes

### Features and benefits

- Rapid
- Sensitive to 25mIU/ml
- Just one step
- Proven accuracy of >99%
- Inbuilt procedural control
- Room temperature storage
- Long shelf life
- Results in 3 minutes
- Detects pregnancy on day of missed period
- Minimal training required
- Reliable
- Confirmation of correctly performed test
- Ready to use. No need to refrigerate
- Helpful for small-volume users


[www.suresteptest.com](http://www.suresteptest.com)

 inverness medical uk

## One-Step hCG Pregnancy Test

### Test Procedure

- 1 Immerse the test strip vertically in the urine specimen, up to the MAX line, for at least 10-15 seconds.
- 2 Place the test strip on a non-absorbent flat surface. Read after 3 minutes. A pink/red line must appear in the control line region (C) to show that the test has worked correctly. Any result that appears after 10 minutes should be ignored.

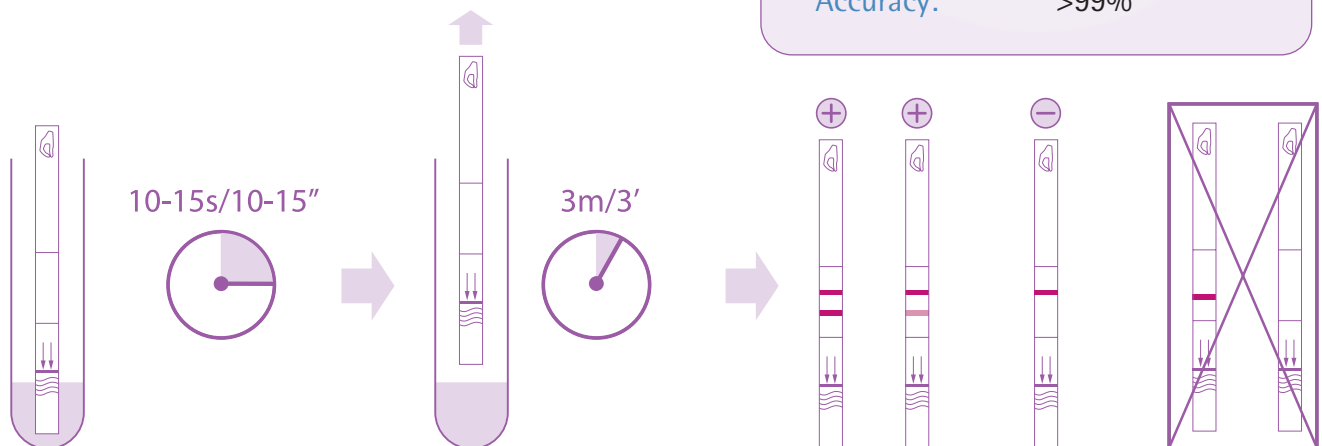

### Product Information




Cat no.	FHC-101 (50T)
Kit size:	50
Shelf life:	24 months
Storage:	2 - 30°C
Sample type:	Urine
Time to result:	3 minutes
Sensitivity:	25 mIU/ml
Specificity:	100%
Accuracy:	>99%

Distributed by



**Inverness Medical has  
changed it's name to Alere**  
0161 483 5884  
ukcustomer@alere.com  
www.alere.co.uk

### Further Products

-  SureStep hCG FHC-102 (40T)
-  SureStep LH FLH-102 (40T)
-  SureStep Drug Screen Test (DST)