

# Cholestech LDX®

## Tips & Tricks

At Alere Ltd we are always trying to think of ways to serve you better.

This Tips & Tricks guide is designed to keep you up-to-date and keep your service running perfectly.



### Cholestech LDX Cassettes

1. As soon as you get a new supply of Cholestech LDX Cassettes put them straight into the fridge (2-8°C)
2. The Cholestech LDX Cassettes have a life of 30 days once removed from the fridge so write on the pouch or box when you have removed them.
3. Once the Cholestech LDX Cassettes have been removed from the fridge, avoid placing them back in the fridge repeatedly as this could affect the reagents inside the cassettes.
4. When you have removed your Cholestech LDX Cassettes from the fridge let them come to room temperature for 10 minutes before opening the pouch.
5. If you have ANY problems or concerns about the use of the Cholestech LDX Cassettes or the reagents please call the dedicated Customer Care team (see contact details below) who will be happy to answer your questions.

**If we don't know you're having problems, we can't help you!**

Following these simple manufacturer recommended steps will ensure the stability required by the devices, helping you; provide accurate results, save money and prevent unnecessary retesting in patients.