


SureStrip[®] hCG (FHC101)


How to Use Guide

The SureStrip[®] hCG detection test is a rapid diagnostic device for the qualitative detection of human chorionic gonadotropin (hCG) in urine. Below is a pictorial of how to use the test. Please observe the usual health and safety guidelines and ensure that the test is suitable for use. Please refer to the instructions for use for full information as this is merely a guide.

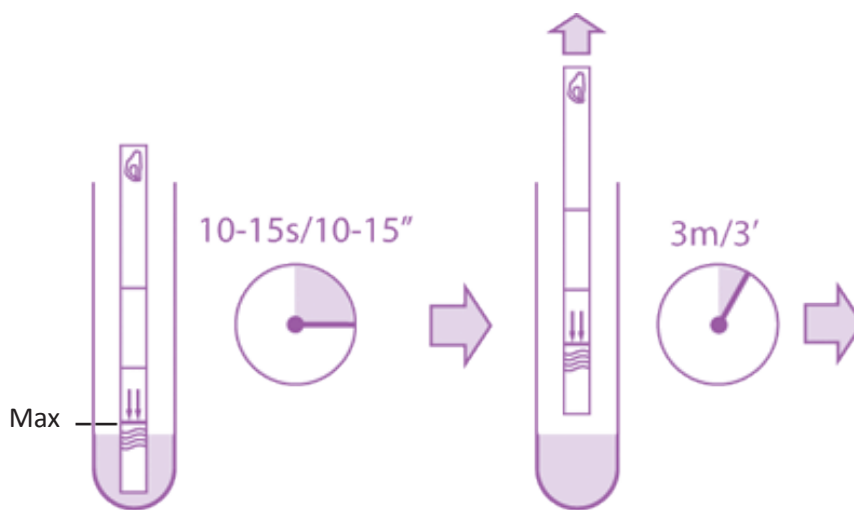

Results

After 3 minutes a line must appear in the control line region (C) to show that the test has worked correctly. The result line will appear in the test line region (T). Any result that appears after 10 minutes should be ignored.

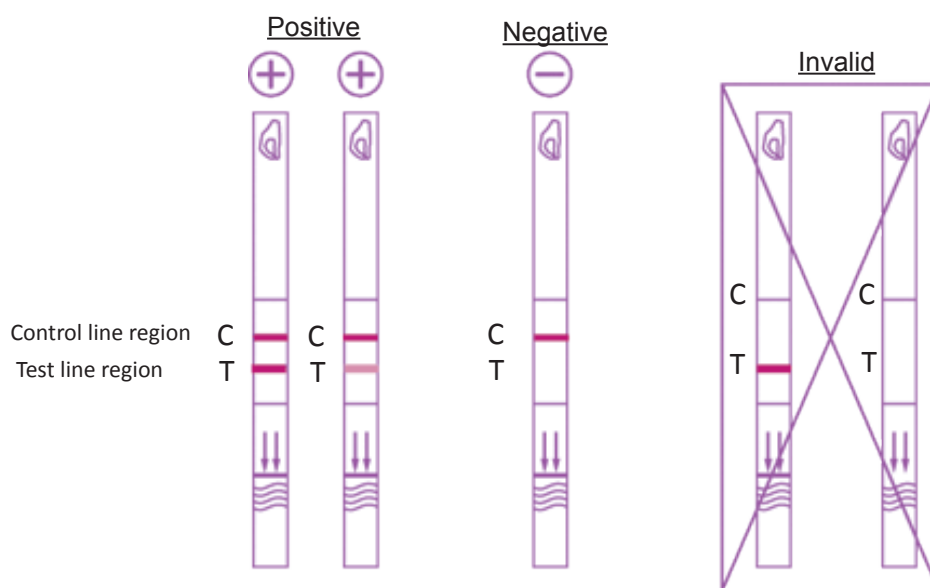

Please see instructions for use for full information.

If you have any further questions please do contact us via the enquiry button on the product page.

For reference hCG positive and negative controls are available (510068). Please evaluate these in-house.