


SureStepTM hCG (FHC102)


How to Use Guide

The SureStepTM hCG detection test is a rapid diagnostic device for the qualitative detection of human chorionic gonadotrophin (hCG) in urine. Below is a pictorial of how to use the test. Please observe the usual health and safety guidelines and ensure that the test is suitable for use. Please refer to instructions for use for full information as this is merely a guide.

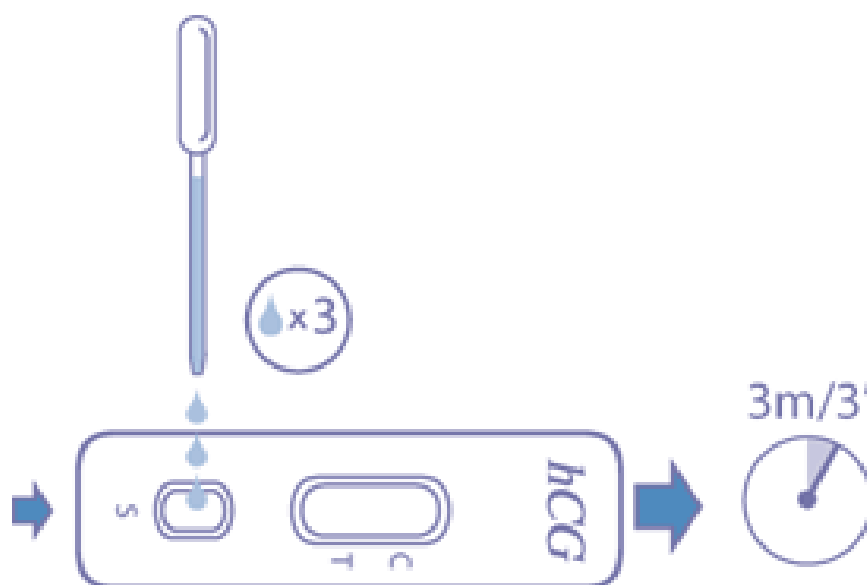

Results

The test must be read 3 minutes after applying the sample. A line must appear in the control line region (C) to show that the test has worked correctly. Any result that appears after 10 minutes should be ignored.

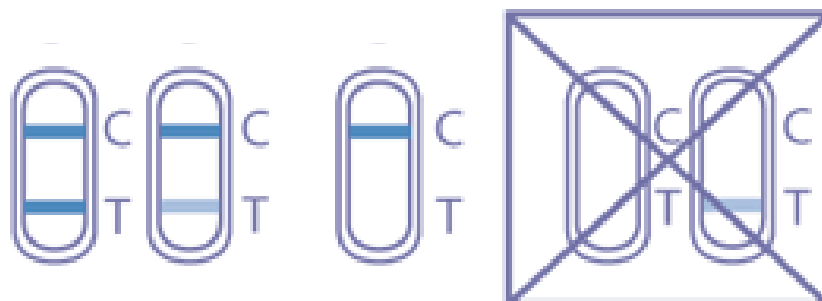

Please see instructions for full information.

If you have any further questions please contact us via the enquiry button on the product page.

For reference hCG positive and negative controls are available (510068). Please evaluate these products in-house