

# Careplan<sup>®</sup> VpH Gloves (505006 & 504102)

## How to Use Guide

The Careplan<sup>®</sup> VpH Glove is designed for measuring the acidity of the vaginal environment (vaginal pH). Below is a synopsis of how to use the test. Please observe the usual health and safety guidelines and ensure that the test is suitable for use. Please refer to the instructions for use for full information as this is merely a guide.

### Step 1

Place Careplan Glove on hand. A test strip is located at the tip of the index finger.




### Step 2

Insert index finger into the vagina. Once inserted the tip changes colour.



### Results

The colour of the test strips indicates the pH level of the vagina. Compare to the colour scale for pH value.

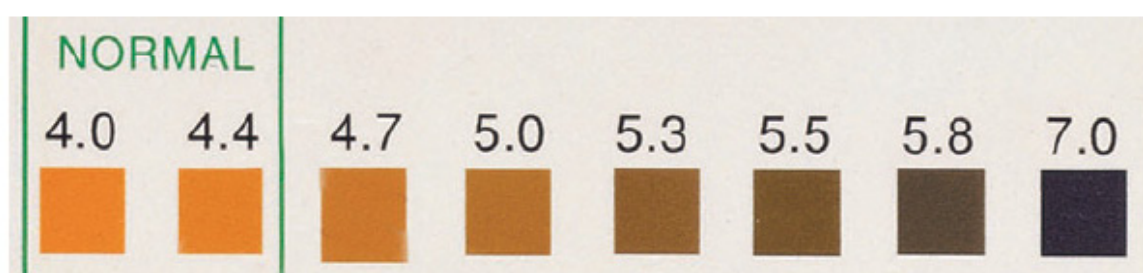

Please see instructions for use for full information.

If you have any further questions please do not hesitate to contact us via the enquiry button on the product page.